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Bruns

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## Efficient Filtering Mask

- Hypoallergenic / Latex Free
- High Filtration Capacity
- Adjustable Padded Nosepiece Provides Comfortable Fit
- Remove the Suspended Particulats
- Dust Pollen Protection

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Please seal tightly after opening, sealing zipper is conducive to the preservation of the product. Please tear along the dotted line (cut) ✂

#### FITTING INSTRUCTIONS:

1. Open the respirator with both hands.
2. Cup the respirator firmly against your face with the nosepiece on the bridge of your nose.
3. Stretch and pull the lower headband over the head and position below your ears. Stretch and pull the top headband on the back of your head above your ears.
4. Press plastic nosepiece to conform snugly around your nose.
5. Seal check
  - To test the fit of a respirator without an exhalation valve cup both hands over the respirator and exhale sharply.
  - To test the fit of a respirator with an exhalation valve, cup both hands over the mask and inhale sharply.
  - A negative pressure should be detected inside the respirator.
  - If air flow is felt in the nose area, re-adjust/tighten the nose clip.
  - If air flow is felt around the edges of the respirator, reposition the respirator/head strap to achieve a better fit.
6. Change the mask immediately if breathing becomes difficult or mask becomes damaged or distorted.
7. Change the respirator if a proper face seal can not be achieved.
8. Careful observance of these instructions is an important step in safe respirator use.

#### **WARNING**

High efficient against particulate aerosols free of oil. This respirator helps protect against certain particulate contaminants but does not eliminate exposure to or the risk of contracting disease or infection. Misuse may result in sickness or death.

#### Guangzhou Harley Commodity Company Limited

Building 16, Huayua Industry Park, Taiping Town, Conghua District, Guangzhou, China  
FAX: 020-37512951  
TEL: 020-37512950  
WEB: www.kn95.net  
Made in China

6 925756 045669

<b>合格证</b> Certificate of conformity	
品牌: 哈雷	型号: L-103V
Brand: Harley	Model: L-103V
品名: 防护口罩	
Product Name: Mask	
数量: 2只/袋	等级: KN95
Quantity: 2pcs/bag	Grade: KN95
执行标准: GB2626-2006	
Standard: GB2626-2006	
产品构成: 无纺布, 熔喷布, 热风棉	
Composition: non-woven fabric, meltblown fabric, cotton	
检验员: 01	<b>检验专用章</b>
Surveyor: 01	
生产日期: 2020.04.15	
Manufacture Date: 2020.04.15	
生产批次: 20200414	
Lot no: 20200414	
有效期限: 3年	
Validity date: Three years	
生产商: 广州市哈雷日用品有限公司	
Manufacturer: Guangzhou Harley Commodity Co., Ltd	
地址: 广州市从化区太平镇元洲岗村 (厂房A01)	
Address: (Plant A01) Yuanzhougang Village, Taiping Town, Conghua District, Guangzhou	